



## Easy Lift

### Harness for lifting the Scoop 65 EXL

(with a hoist for transferring disabled persons in accordance with the UNI EN ISO 10535 standard)

The Easy Lift harness allows patients to be simply and securely lifted short distances using the Scoop stretcher. It is useful for transferring the patient from a stretcher to a bed or to an examination table.



It is very easy to attach and remove, does not require any special skills and can be used with all Scoop 65 EXL stretchers.

#### Measurements and features

Length:.....1250 mm / 1150 mm  
Width: ..... 15 mm  
Weight: ..... 2.3 kg  
Maximum load: ..... 227 kg  
Tensile strength per webbing: ..... 22 kN

#### Materials used

Webbing: .....PE HT

#### Features

2 long PE HT fibre webbings (1250 mm)  
2 short PE HT fibre webbings (1150 mm)

#### Static tests in accordance with UNI EN ISO 10535

Static tests have been carried out on the Easy Lift system in the laboratory in order to determine the maximum load of each of webbing, the toughness of the materials and the seam strength.

The Easy Lift system was tested in conjunction with the FERNO Scoop 65 EXL and with the spreader bars of hoists:

- BHM Medical Inc. for Arjo's Maxi Sky 440
- GULDMANN Standard Ceiling Hoist

#### Cleaning

wash in lukewarm water (max 30°C), use a neutral detergent and disinfect with disinfectants that are compatible with the materials used (PEHT).

#### Disinfecting

Leave the Easy Lift to soak for one hour in water (max 20°C) with disinfectant, then rinse with clean water.

Part No. **PS0765EXL**

**Meets the requirements for the  
93/42 / EEC (Class I Medical Devices)  
EN ISO 10535 (Dec-2006)**

